

quick start guide



.01 download app



For iOS, go to the App Store®.




For Android™, go to Google Play.

Search for “LifeEar” and install the app. Open and follow the instructions to get started.

note: The app is required to personalize your LifeEar.

App Store is a service mark of Apple Inc., registered in the U.S. and other countries. Android, Google Play and the Google Play logo are trademarks of Google Inc.

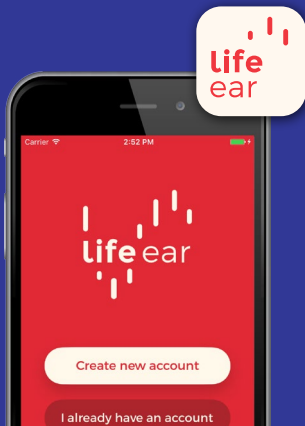



.02 insert battery/turn on

Remove tab to activate battery.


Wait one (1) FULL minute. Use

nail grip to open battery door and insert battery “+” side up.

Close battery door to turn
CORE on.

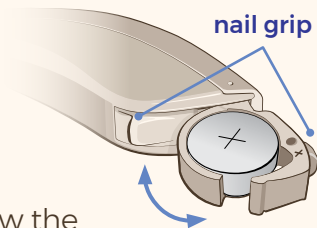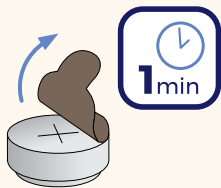


Follow the
app instructions
to connect CORE
and your device.

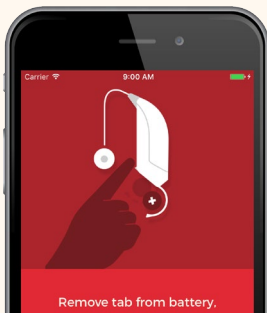


.03 personalize

Insert *ComfortTIP™* with gentle pressure into ear canal. Place the **CORE** body behind ear. Tuck stabilizer bar in the outer bowl of ear.

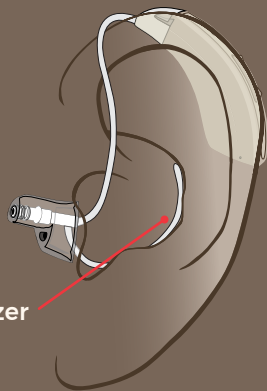


stabilizer
bar

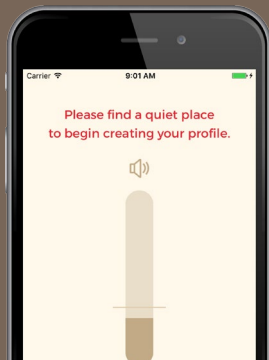

Find a quiet place and follow the instructions to personalize your **CORE**.


.04 volume/program

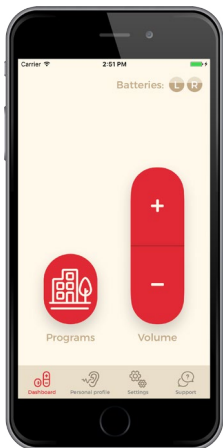

Use the app Dashboard to change volume and program.
Or use the hearing aid button:
volume – press/quickly release
program – press/hold

**raises volume
and program**

**lowers volume
and program**

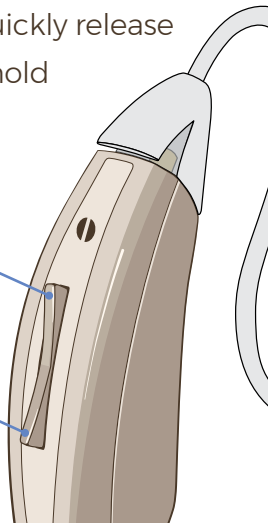


.05 regular maintenance

Wipe daily: Use the microfiber cloth and cleaning tool brush to clean your CORE.

Use cleaning tool wire regularly:

Remove tubing, thread wire through tubing and tip, and remove debris.

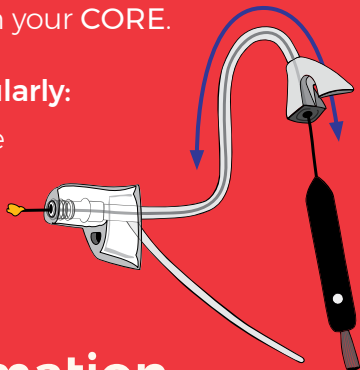

for more information

Please refer to your user manual and app manual included in your LifeEar box.